


The Reliable and Rapid



Dx S Typhi IgG/IgM




**Point of care. Anytime.
Anywhere.**

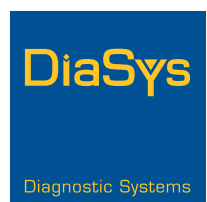

CHOOSING QUALITY.

Rapid & Reliable Two Step Assay

Step 1

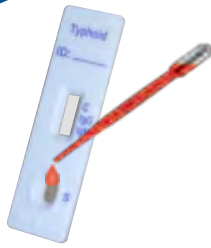

Using dropper, add 2 drops of whole blood or serum or plasma specimen into the sample well

Step 2



Add 1 drop of QDx S. typhi clearing buffer

Step 3



Read the test result in 10 minutes



Easy to Interpret

Positive



Positive




Positive



Negative



Invalid

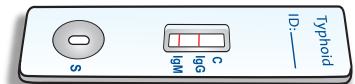


Invalid



Kit Specification

Test Type	Rapid Qualitative Assay
Sample Type	Whole Blood / Serum / Plasma
Sample Volume	2 Drops (approximately 50 μ L)
Test Principle	Lateral Flow Chromatography Immunoassay
Read Time	10 Minutes
Relative Sensitivity	92.9%
Relative Specificity	99.3%
Storage Temperature	2°C - 30°C

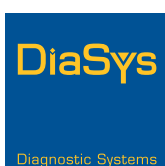

DiaSys Diagnostics India Pvt. Ltd.

Plot No. A -821,TTC, MIDC,
Mahape, Navi-Mumbai, 400 710
Maharashtra, INDIA
www.diasys.in

Call Us On

Toll free No INDIA

1800-200-1120

For Calls out of INDIA

+91 22 33 714344

Email Us at

info@diasys.in

